232mm

JD-Hook® Safety Stylet/Trocars
Chest Tube/Catheter Placement and Tunneling device

Instructions for use

Intended Use

The JD-Hook® Safety Stylet is a blunt tip trocar designed for Subcutaneous tunneling needed for proper placement of various devices including vascular catheters and drainage catheters. The blunt tip and safety angulation design reduces the risk of deep tissue injury and provides directional control for directing the devices into appropriate position.

Contraindications

- Active bleeding
- Bleeding disorders
- Coagulation disorders
- Unstable cardiovascular status
- Severe respiratory distress
- Penetrating chest and abdominal trauma
- Open pneumothorax
- Tension pneumothorax

Warnings/Precautions

- Immunocompromised patients
- Patients with diabetes
- Patients with liver disease
- Patients with kidney disease
- Patients with heart disease
- Patients with a history of allergies
- Patients taking anticoagulants
- Patients taking antiplatelet drugs

Indications for use

The JD-Hook® Safety Stylet is indicated for Subcutaneous tunneling and access to body cavities for placement of catheters, drainage tubes into appropriate position.

Pleural Cavity Catheter Placement:

1. The JD-Hook® Safety Stylet is inserted into a closed tip chest tube (or the designated JD-1000 chest tube at insertion point located 12.5 cm from the distal tip of chest tube).
2. Once loaded, the stylet tip lies within the blunt radiopaque tip of the chest tube.
3. Insert the stylet loaded with chest tube, at the prepared incision site, at the appropriate intercostal space, entering the pleural space at the upper margin of the lower rib. (Refer to figure 1 for insertion technique)

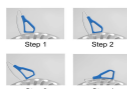
Fig.1: Pleural space insertion technique schematic diagram.

4. Advance the stylet until the operator feels a pop, which indicates that stylet has entered the pleural space.
5. Advance the stylet into the chest cavity till the proximal curved end of stylet is at the skin level.
6. Holding the stylet in place, Advance the chest tube into the pleural space by sliding it over the stylet into the pleural cavity. Advance the chest tube till all the drainage holes and the access point are located within the pleural space.

1

Reg No: 3024792425
Device Listing Number: D524305

100mm

7. Slowly retract and remove the JD-Hook® Safety Stylet, and proceed with management of the chest tube per institutional protocol.

Subcutaneous Tunneling Procedures:

JD-Hook Safety Stylet can be used for making a subcutaneous tunnel between two access points which are up to 12 cm apart. The stylet can be used independently or as loaded over an appropriate drainage catheter (eg, JD-3000, tunneled Long term drainage Catheter).

1. The JD-Hook® Safety Stylet is inserted at the prepared skin incision site at access point
2. The Stylet is then advanced under the skin towards the desired catheter exit site, which is located within 12 cm from access point.

3. Stylet tip is pressed outwards at the Point 2 Skin incision site till the tip is visible at the incised skin surface.

Components

The JD-Hook® Safety Stylet packaging contains following components:

- 1 JD-Hook® Safety Stylet
- 1 Instructions for use Manual

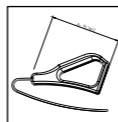
Fig 2: JD-Hook® Safety Stylet

Customer Service

If you have any questions or concerns about the use of the JD-HOOK Stylet, please contact at:

JFD-PK Medical LLC

4400 Cumorah Ave, Center Valley PA
18034 Phone: 484-602-8753, Fax:
484-968-8074

Email: jfdpkmedical@gmail.com

Adverse Event reporting: AER@jfdpkmedical.com

Registration Number: 3024792525

Hangzhou Fushan Medical Appliances Company Ltd.

No. 1288 South Jinxi Road, Linglong Industry park, Lin'an,
Hangzhou, Zhejiang, CN 311301
Registration # 3006787036; FEI # 3006787036

eIFU can be found at www.jfdpkmedical.com/eIFU

SYMBOLS		

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